





Origami Face Shield Assembly Instructions

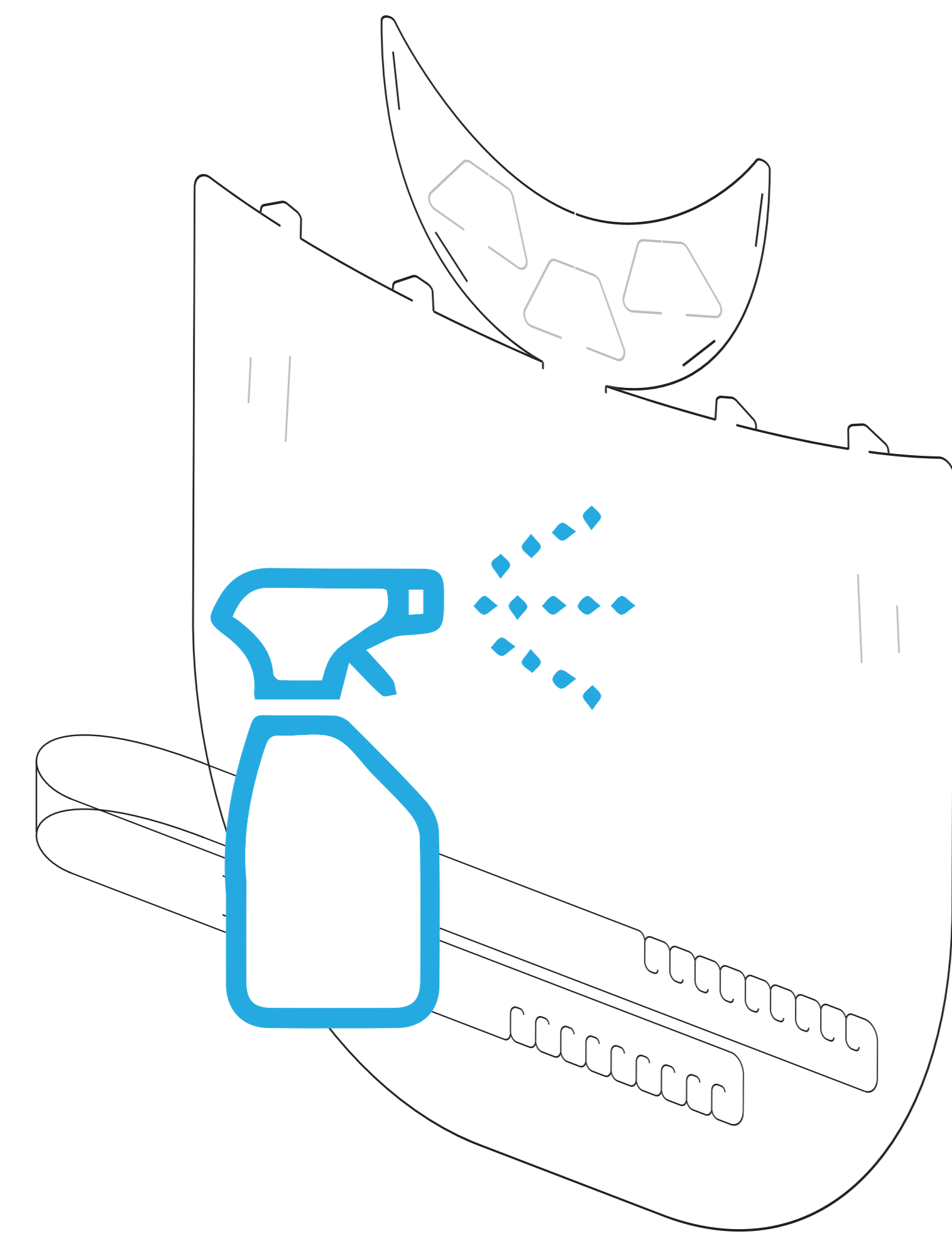



Assembly Video

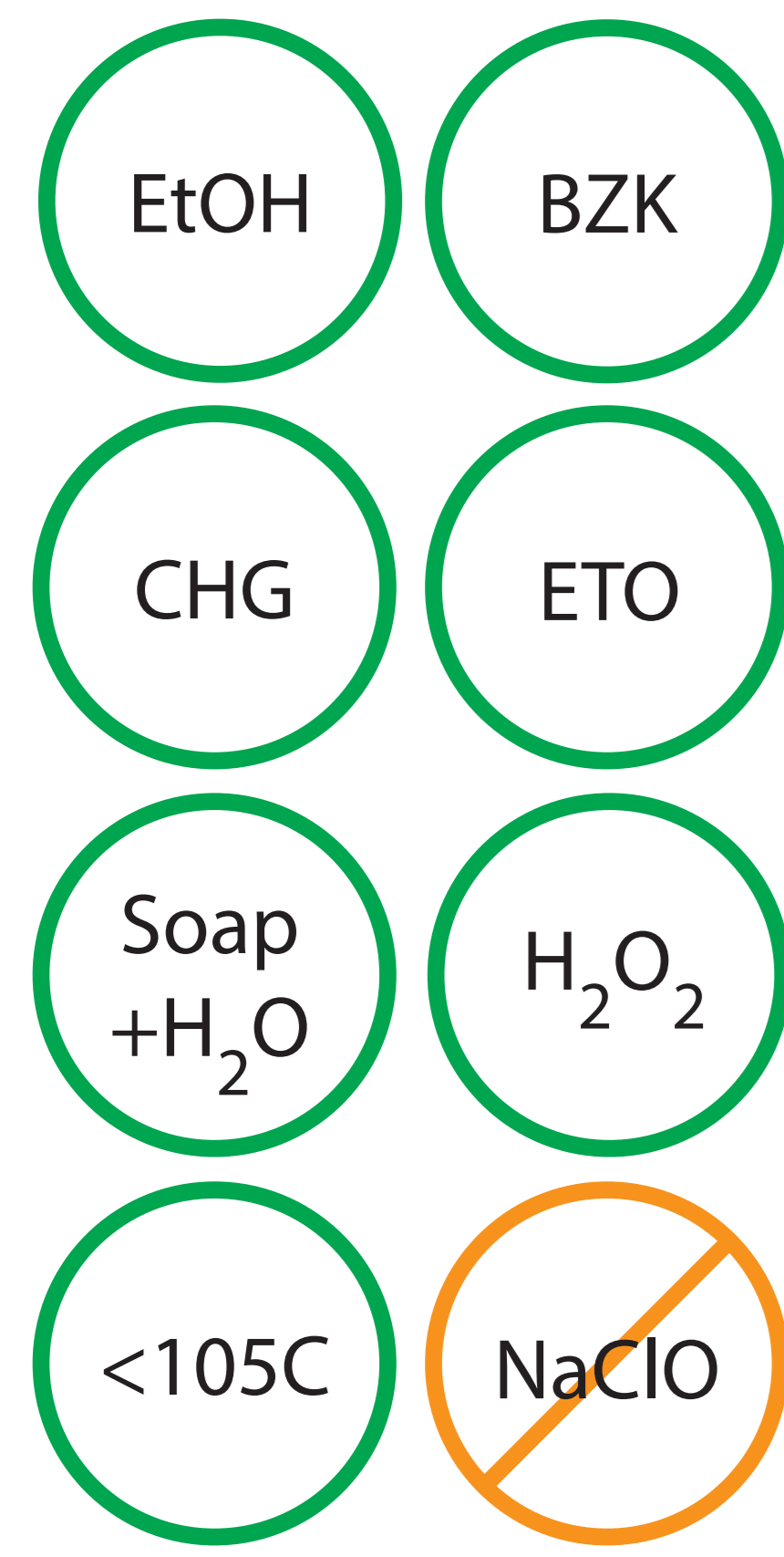






1. Clean and disinfect all parts as needed. Dry.

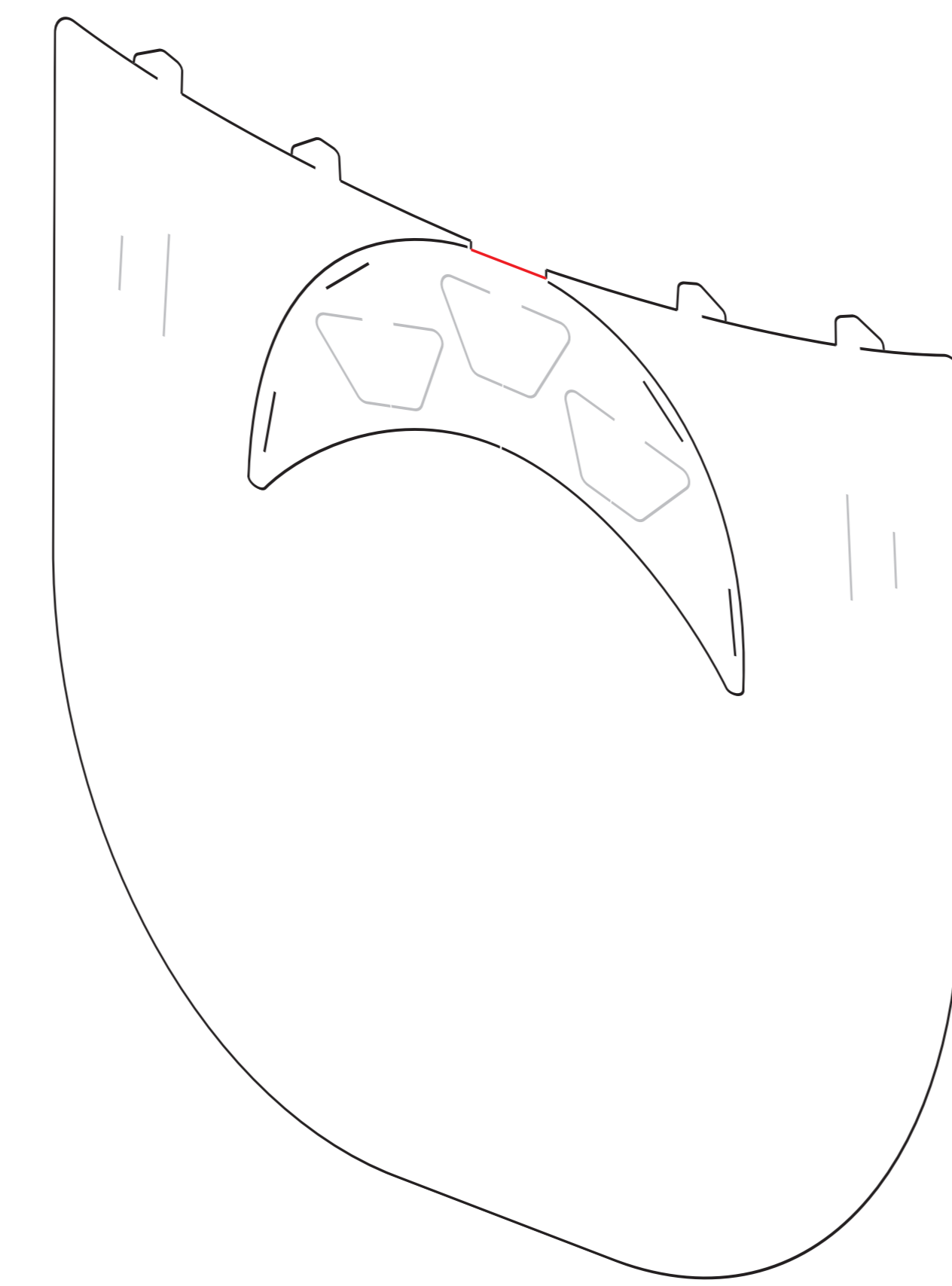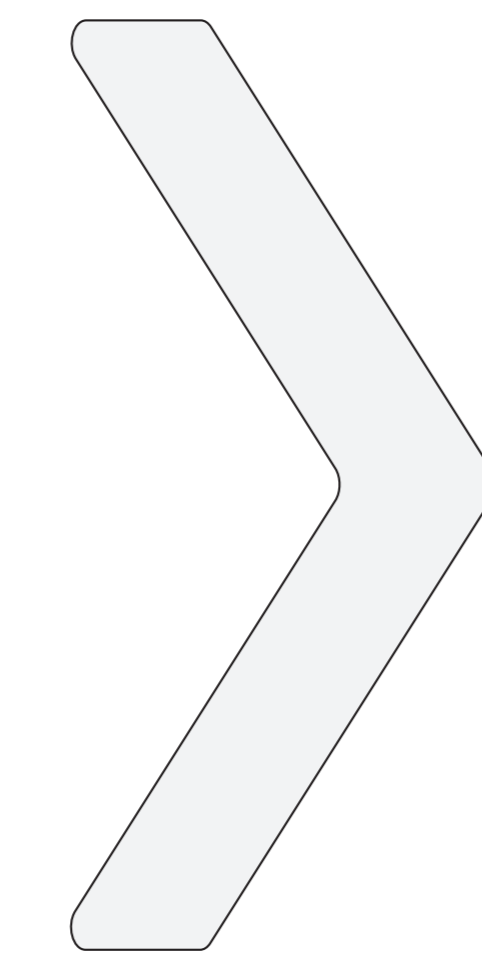



2. Fold the crown piece along the line and crease sharply

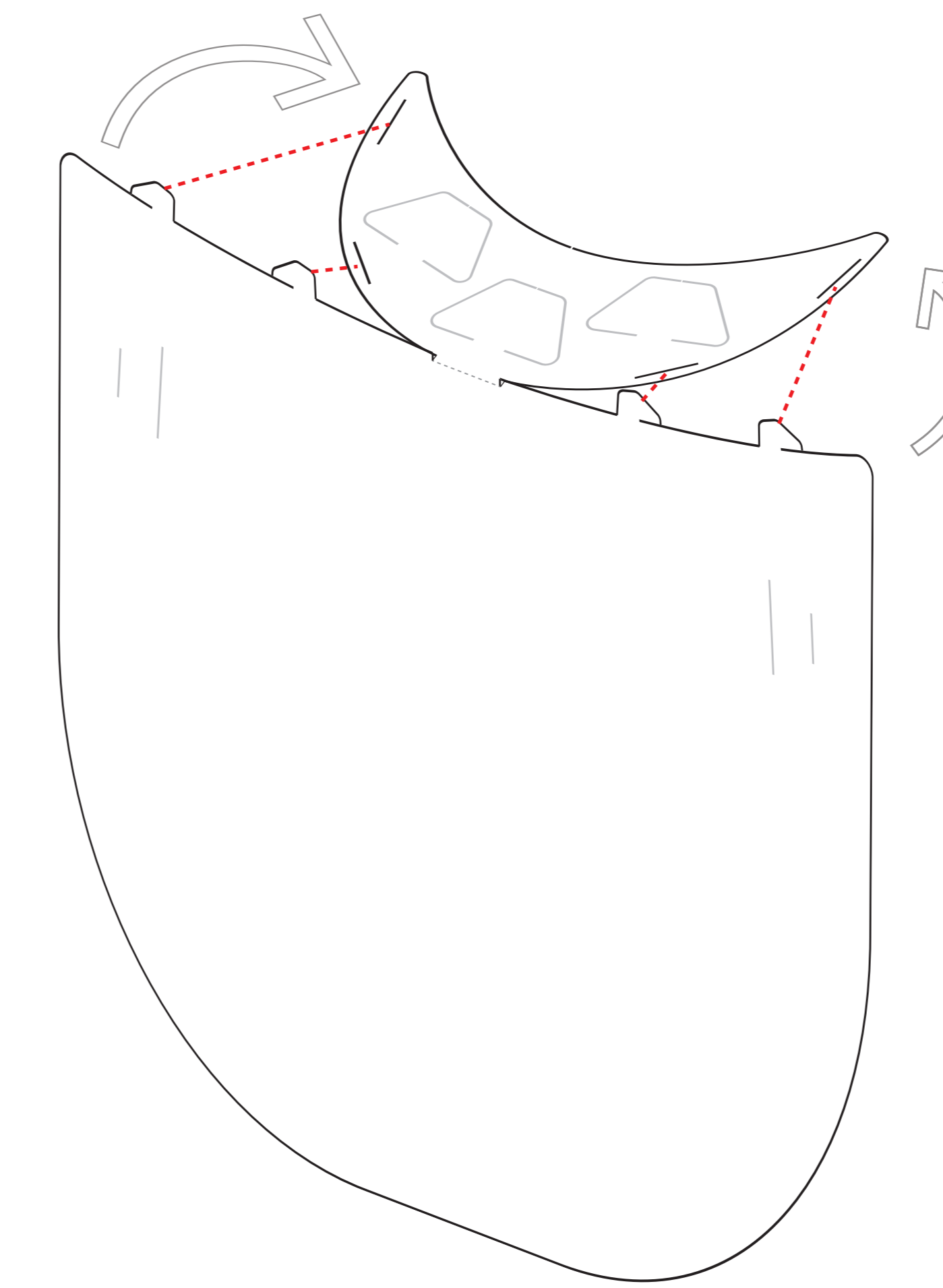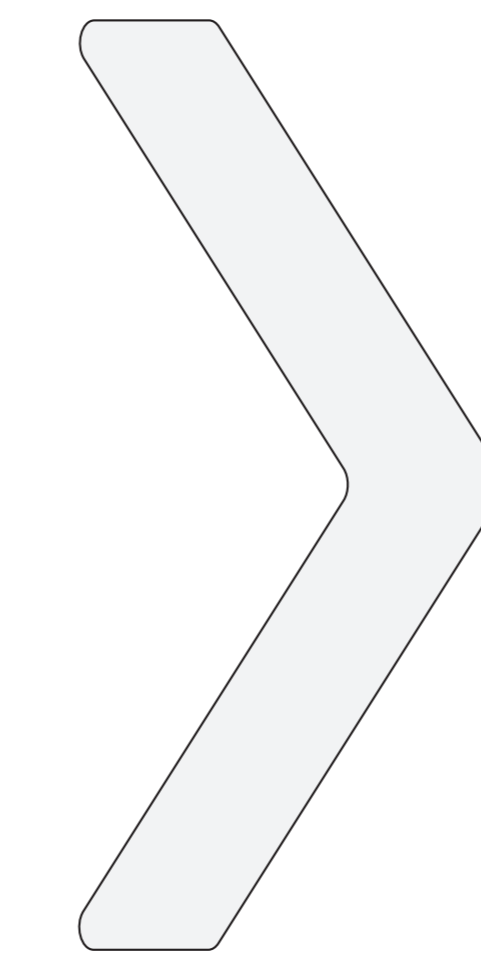



3. Insert the tabs through the slots on the crown piece

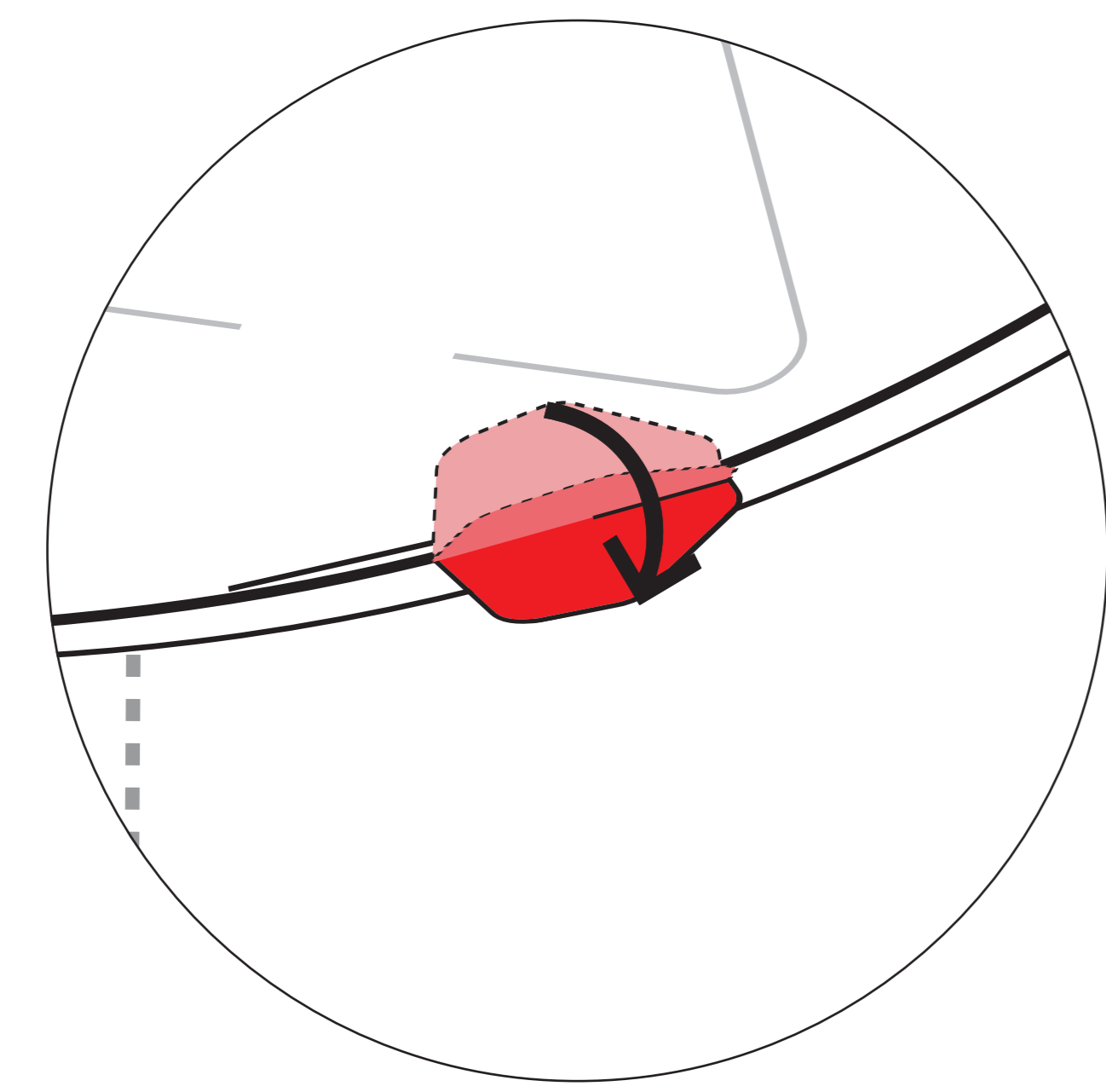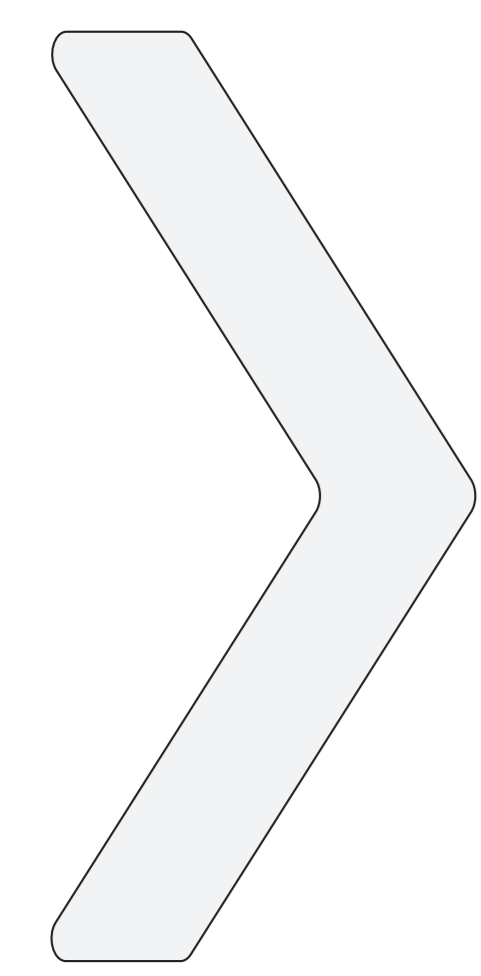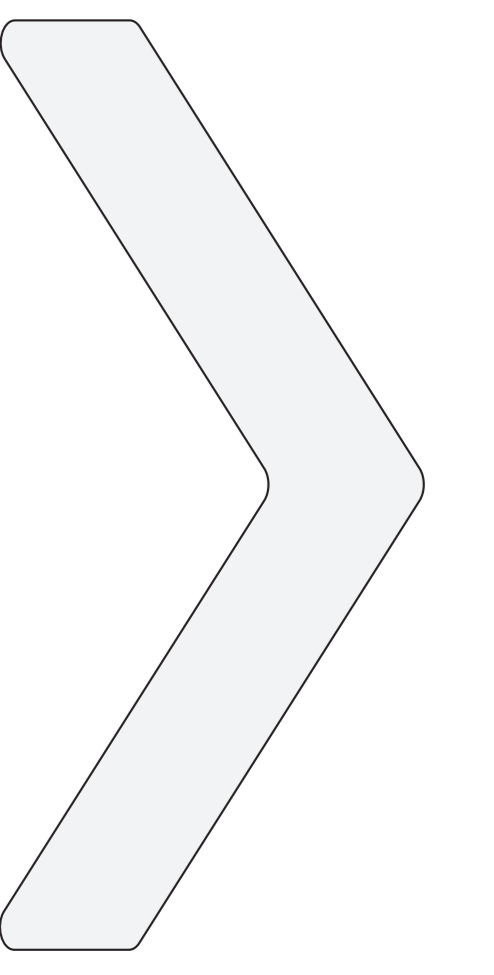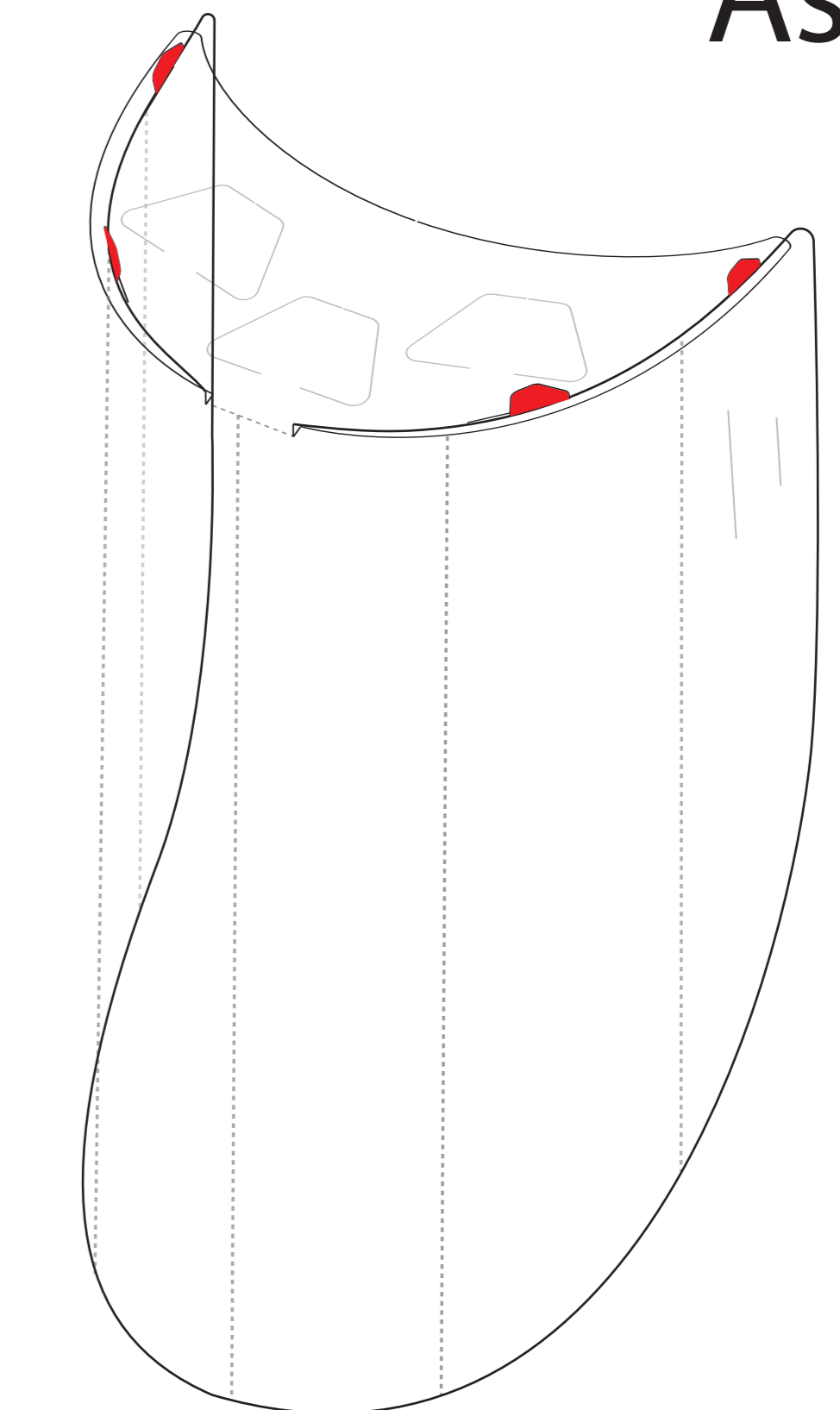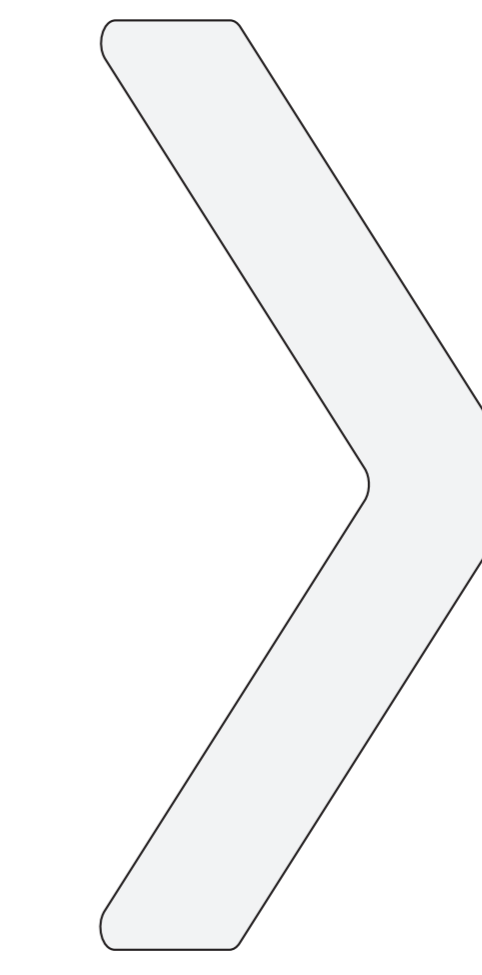



4. Fold the tabs downward and pinch to crease

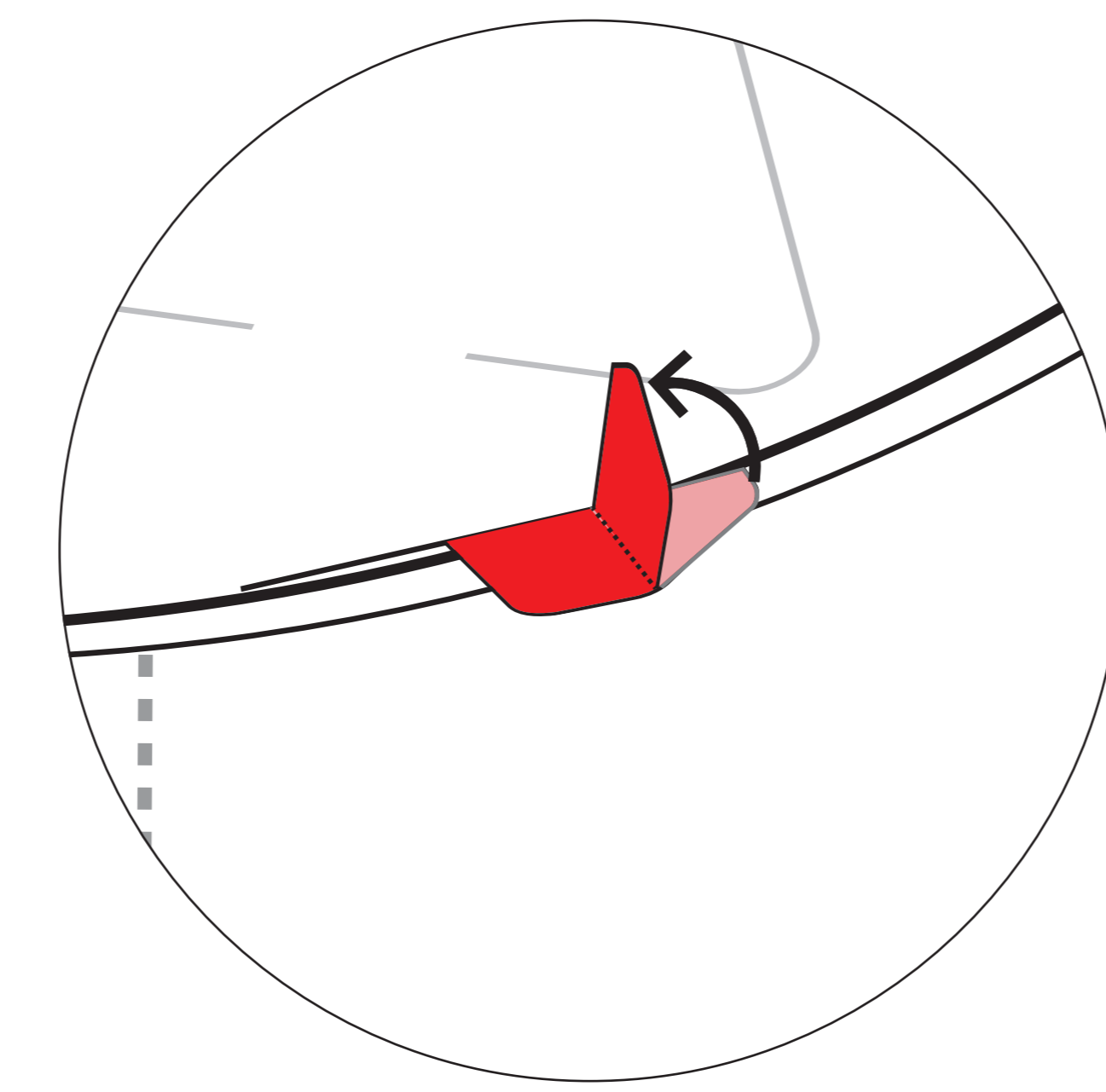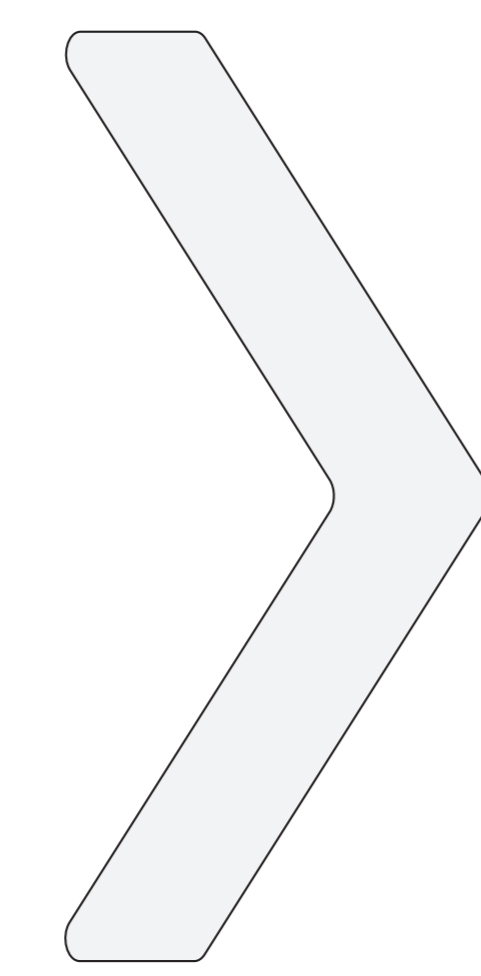



5. Fold the loose end of the tabs upward and crease

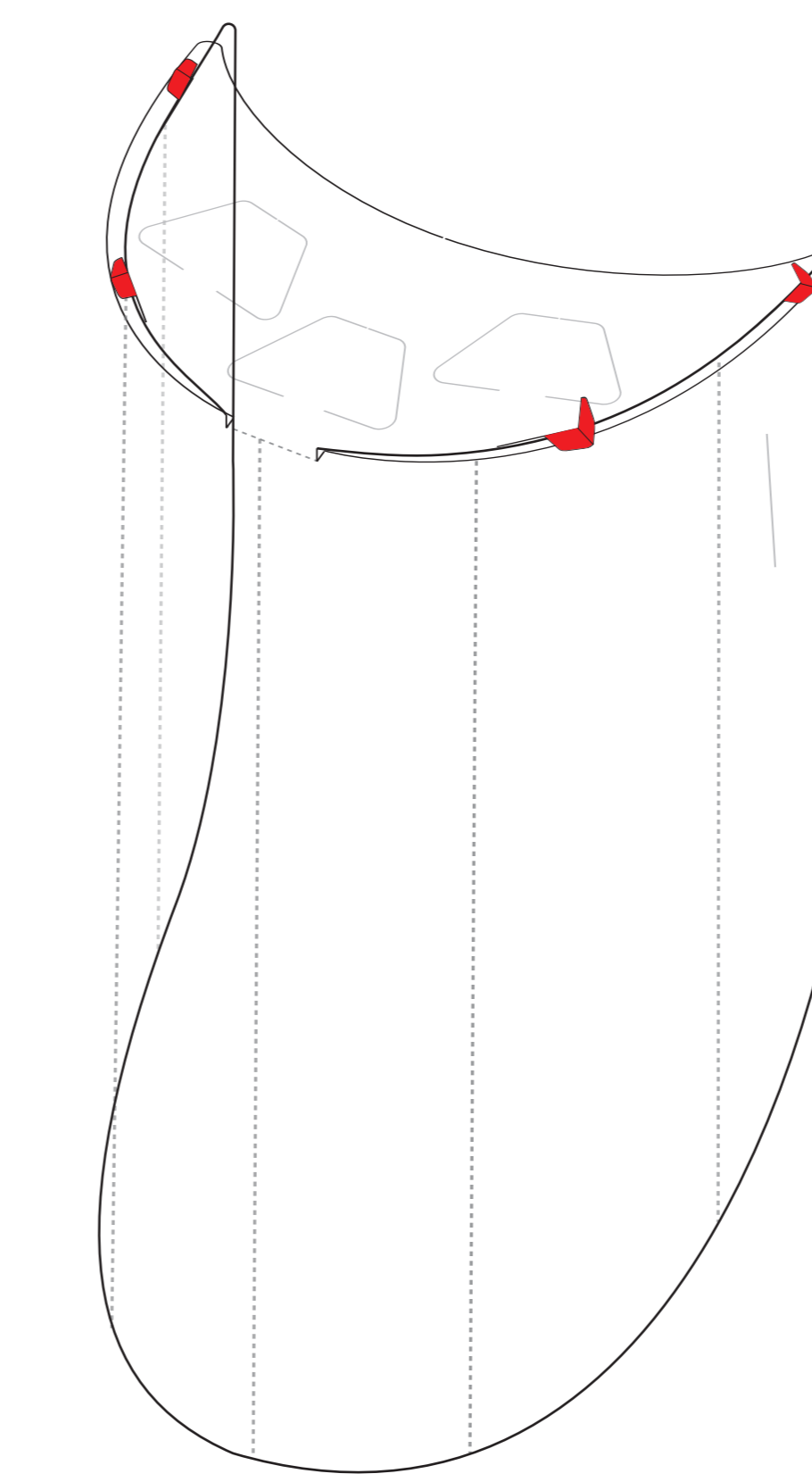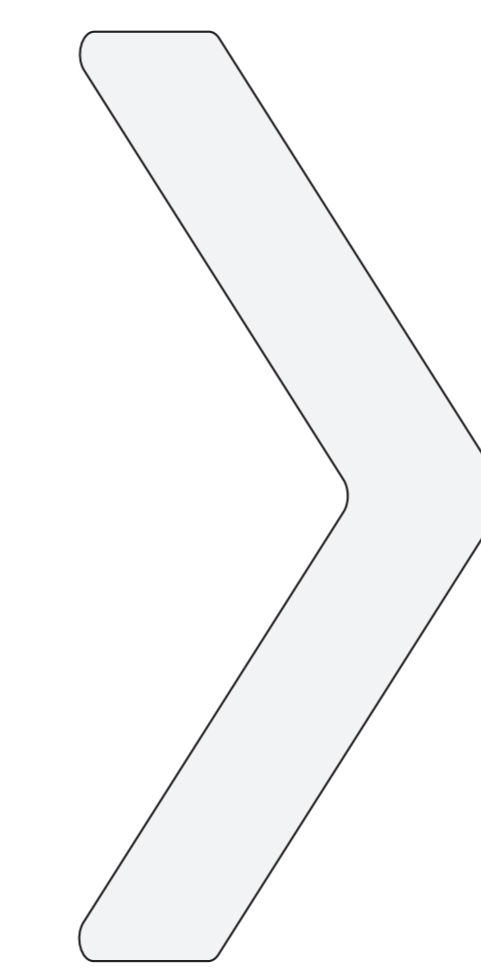



6. The shield piece is now completed.

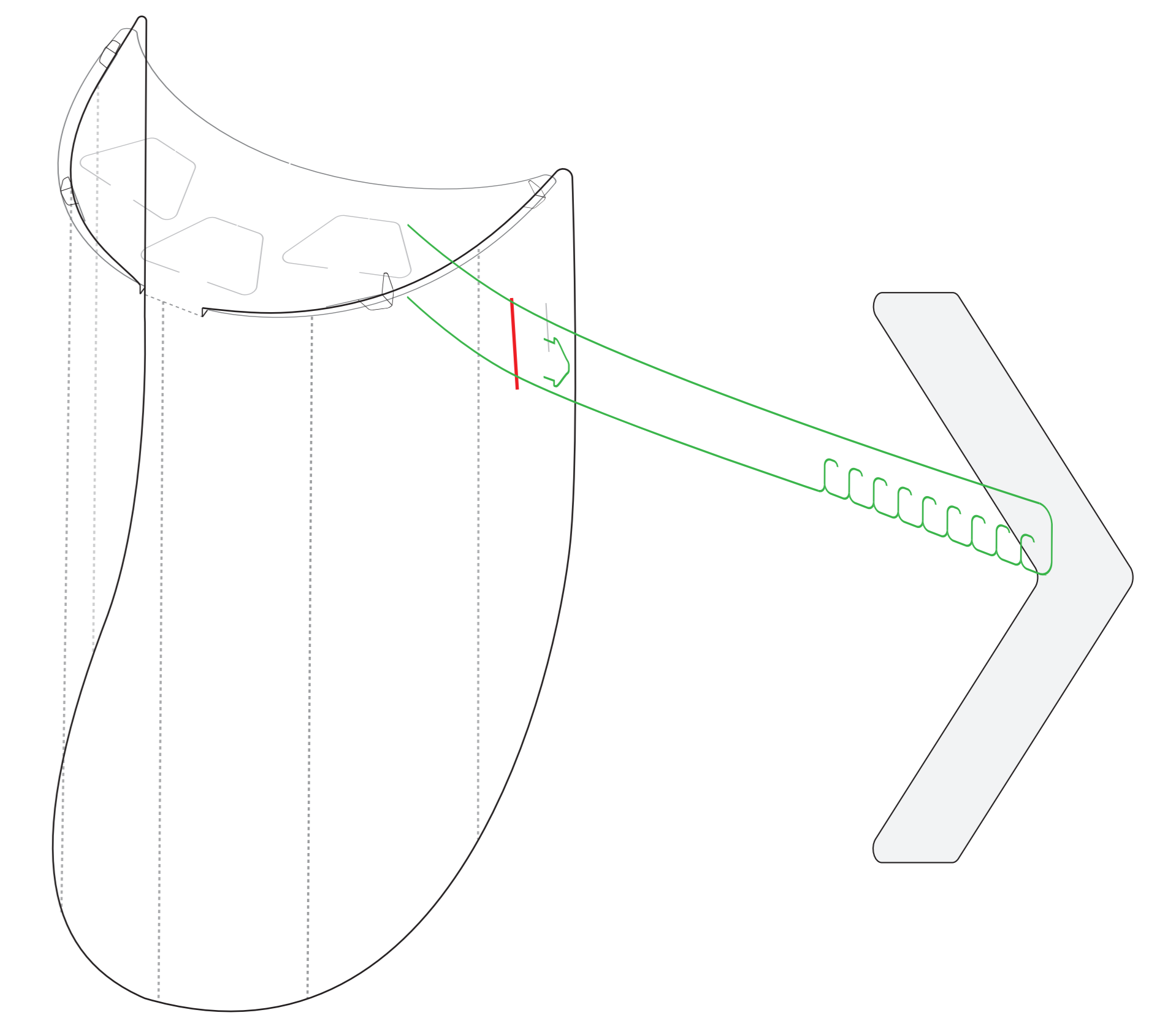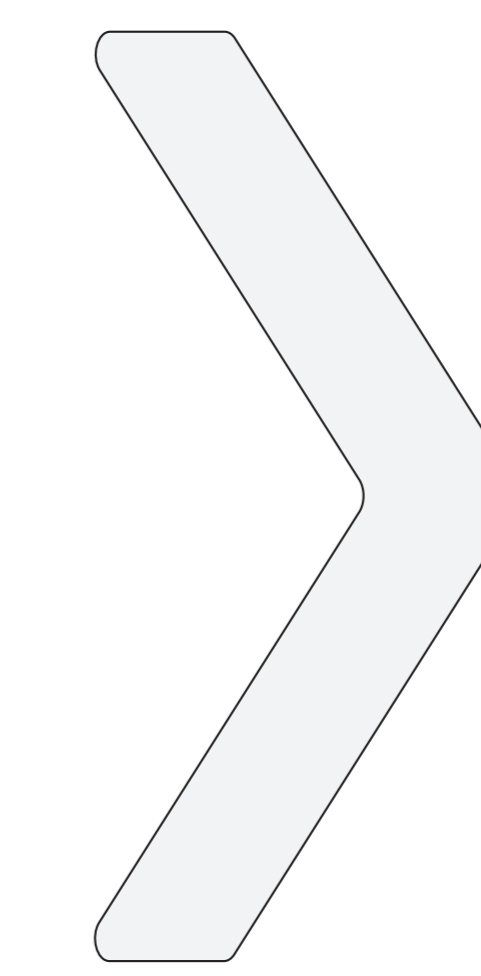



7. Pass the headband through the large slit, from inside to outside, until the arrow-shaped tab comes through

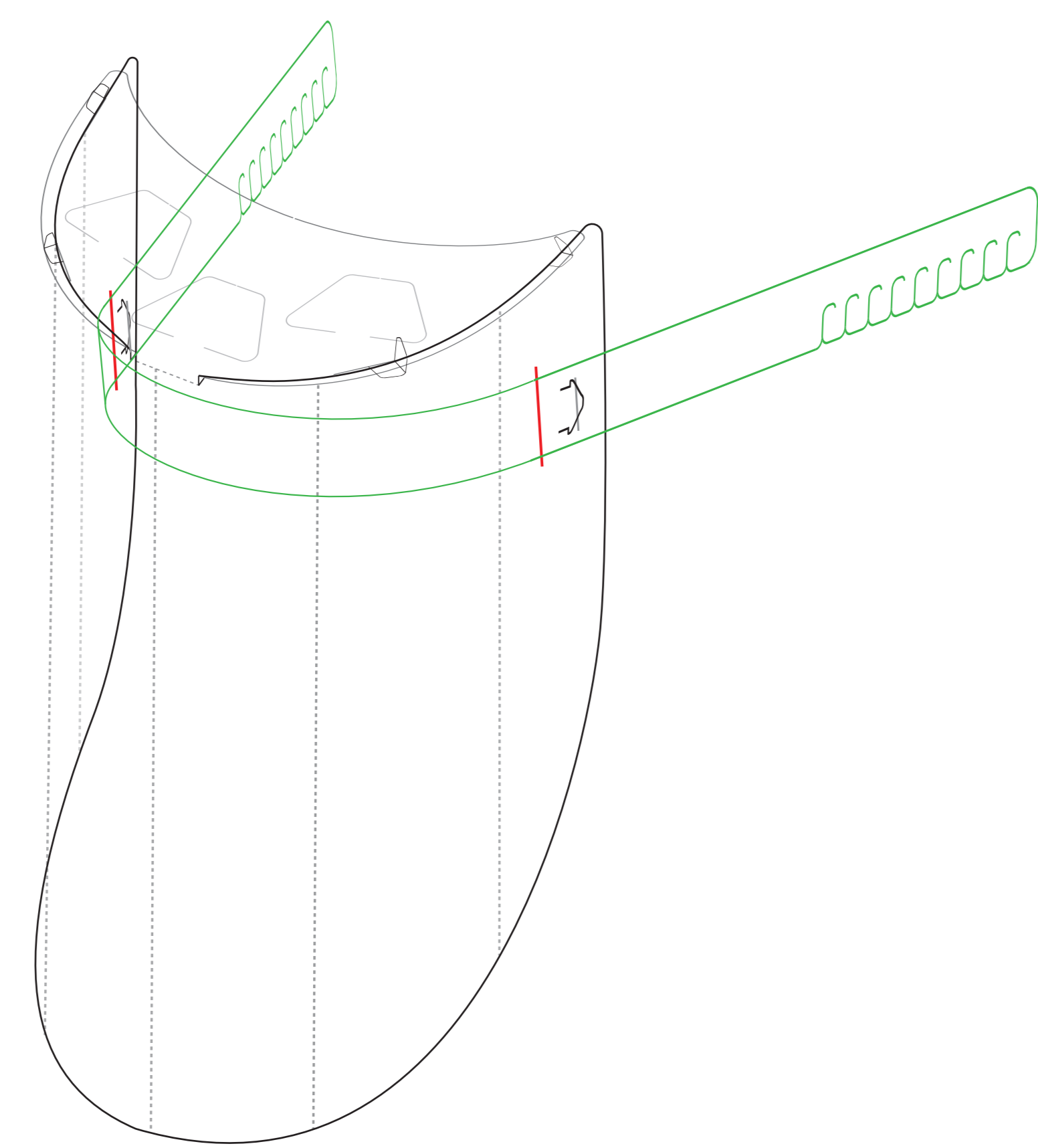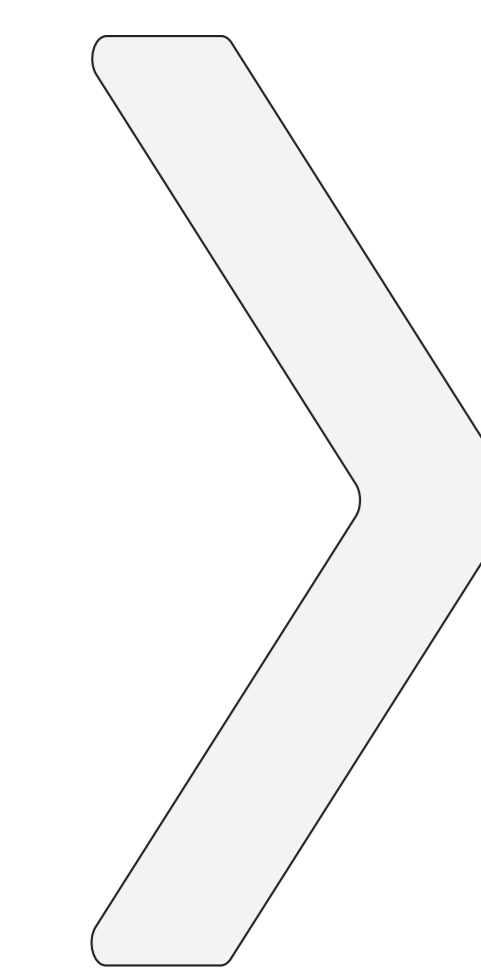



8. Repeat on the other side

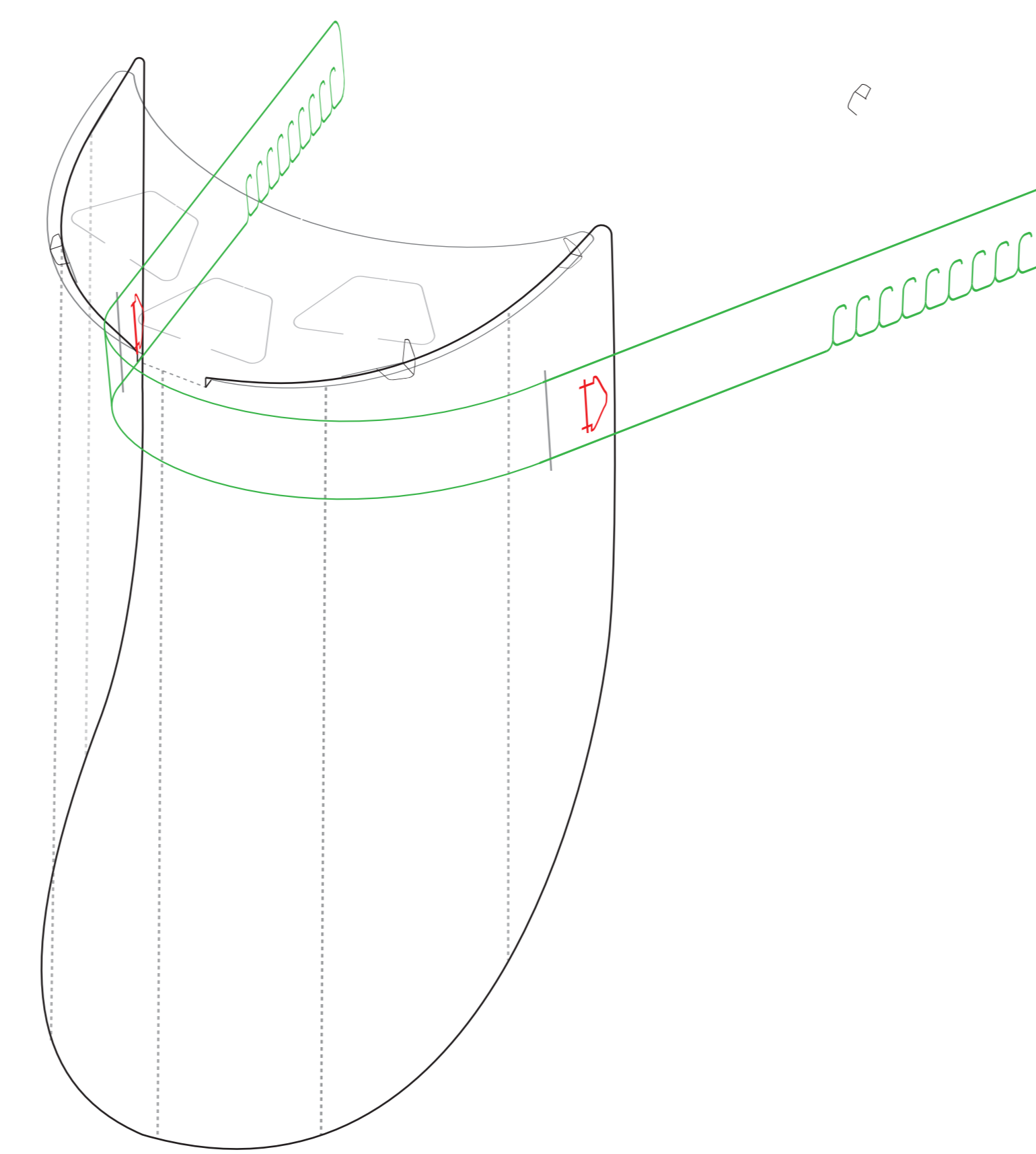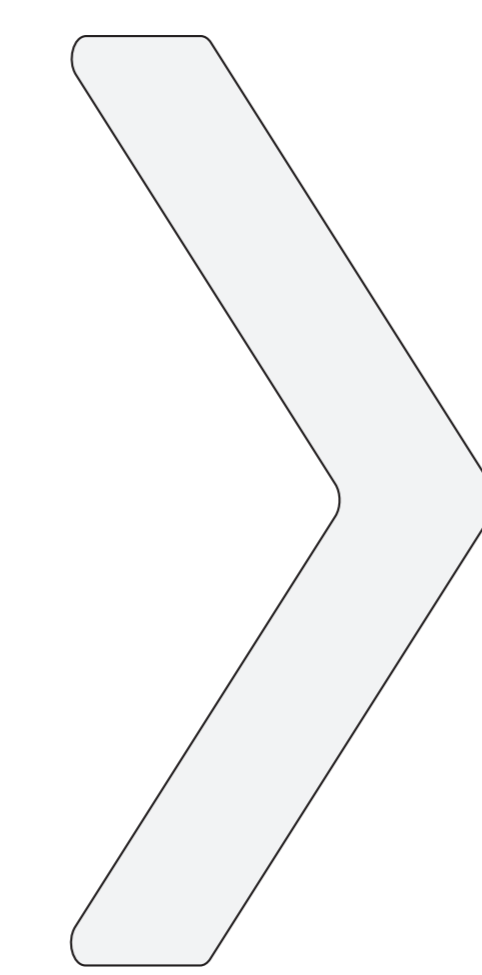

9. Pull arrow-shaped tabs through the small slits until they pop through

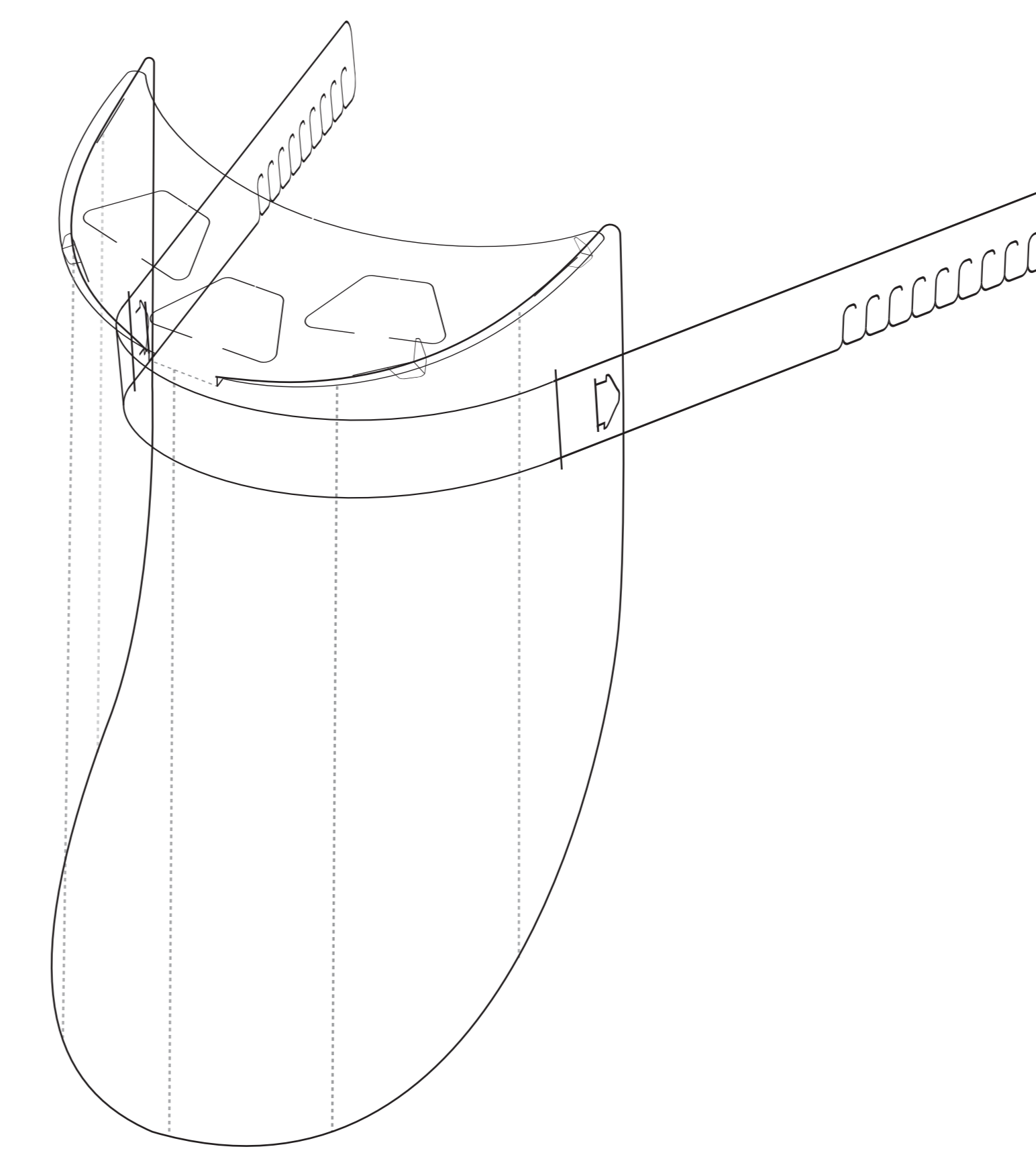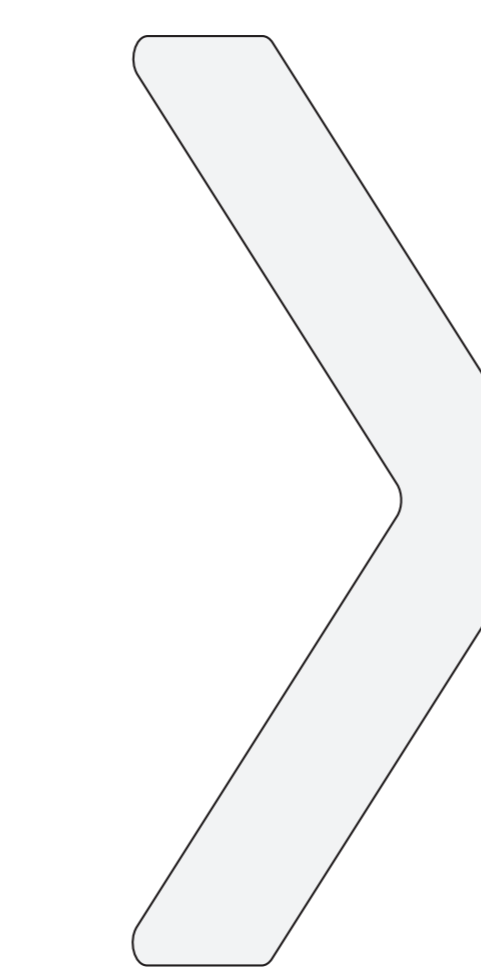

10. Face shield is ready for use

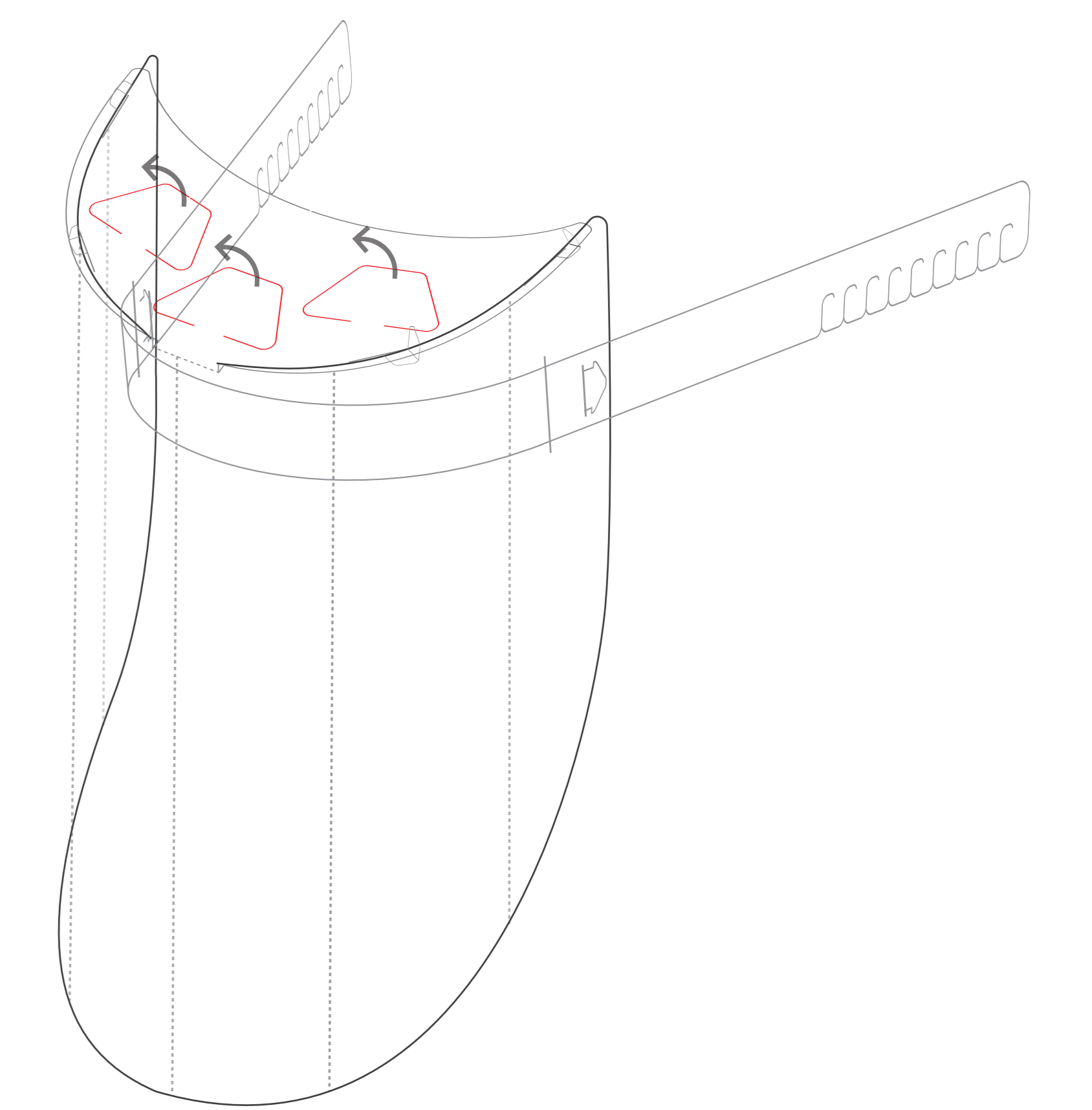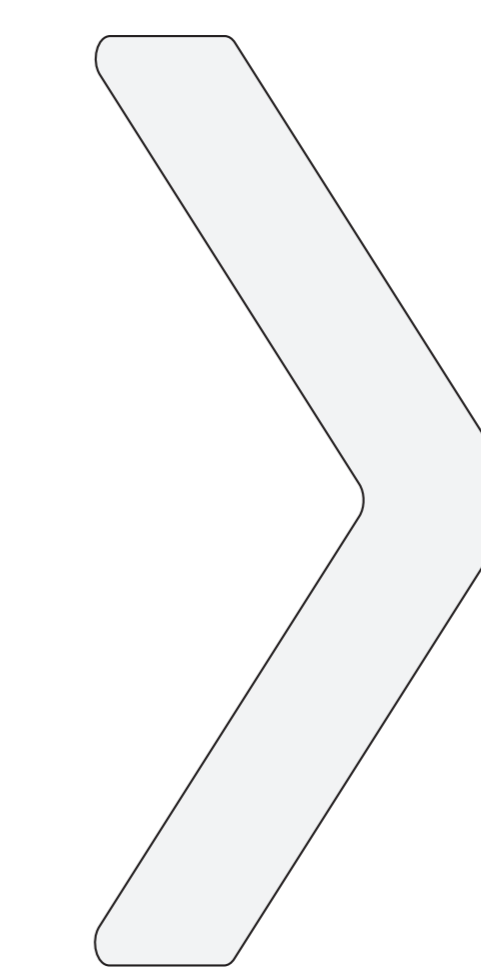

11. Adjust vent flaps to the desired position